Quick & Reference Guide

IMPORTANT!

- FIRST**, we strongly recommend you read all instructions carefully and watch the brief instructional video on our website at ptsdiagnostics.com. **THEN**, follow all steps as you perform the test.
- Do not** open pouches until instructed.
- Make sure kit number on analyzer matches kit numbers on pouches.
- Use indoors between 18°C - 25°C (64°F - 77°F) and out of direct sunlight.
- Do not use if dot on the temperature label of box is red or if the product is past the expiration date.
- If the kit has recently been opened at temperatures outside the recommended conditions, keep the kit at room temperature for at least one hour before use.

1 Open Plastic Shaker Pouch*

Do NOT open foil cartridge pouch until Step 6.

2 Stick Finger

- Wash hands or clean with alcohol and dry before testing.
- Remove cap and press lancet into finger.
- Gently squeeze finger to get adequate blood.

3 Collect Blood

- Gently touch blood collector to tip of blood drop to fill.
- Make sure collector is adequately filled.

4 Insert Blood Collector

- Fully insert blood collector into shaker body.
- Push hard while twisting to fully insert.

5 Mix

- Shake vigorously 6-8 times (about 5 seconds). This will mix the blood with the solution.
- Stand shaker on table while preparing cartridge.

6 Open Foil Cartridge Pouch*

AFTER Blood Collection

Remove cartridge into analyzer immediately and use within 2 minutes.

7 Insert Cartridge

- Click test cartridge into place.
- Analyzer and test cartridge codes must match.
- If codes do not match, call Customer Service at +1-877-870-5610 (toll-free in US and Canada) or +1-317-870-5610 (direct) to walk you through the test.

8 Prepare Shaker

Remove shaker base.

9 Dispense Sample into Cartridge

- Insert analyzer in seal surface.
- Push down completely to dispense diluted sample and remove in a continuous motion.
- Remove sampler from cartridge quickly. "RUN" will appear.

10 5 Minutes to Results

Display reads "OC" (out of control).

11 Dispose of Cartridge / Save Analyzer

- Dispose of cartridge in your household waste.
- Save analyzer for additional tests. Analyzer will display, for example, "OTL" if there is one test left and "WTE" if you just used your last test.
- Record your A1C result in the result log in Step 12.
- Analyzer will show result for 15 minutes and will turn off automatically.

12 Log Result

Chart your progress here.

Record your A1C results here, and bring this with you when you see your doctor. Mark down your A1C goal in the space provided.

Result Interpretation

- Do not take any decision of medical healthcare without first consulting your healthcare practitioner.
- When this device is used for monitoring of an existing disease, adjust the treatment **only** if you have received the appropriate training to do so.

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